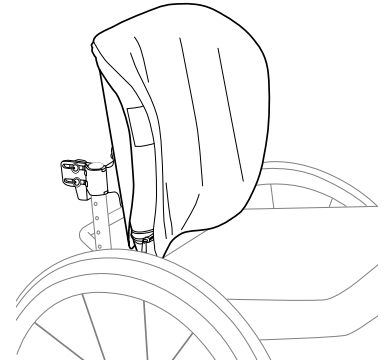
# Icon™ Back System JUNIOR Back Support

WITH VARILOCK™ HARDWARE

## Installation Instructions

*This manual explains how to install, operate and maintain VARILITE back supports. If you do not understand this instruction manual or if you have any questions, please contact VARILITE customer service.*

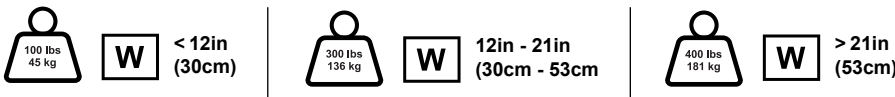
### Product Covered in this Manual

Icon™ Back Low, Icon™ Back Mid, Icon™ Back Tall, Icon™ Back Deep, JUNIOR Mid Back and JUNIOR Deep Back Supports

### Intended Use and Contraindications

The Icon and JUNIOR back supports are designed for wheelchair users and are intended to be fitted to wheelchairs. These back supports provide support and positioning to the trunk and pelvis and reduce pressure on the spine.

For use with round canes 1 inch (25 mm), 7/8 inch (22 mm) or 3/4 inch (19 mm) in diameter.

### Important Warning and Safety Information

VARILITE products should be selected, configured and prescribed by a seating professional who is experienced in seating and positioning, and is capable of determining if the product is suitable for an individual's specific needs. The intended use statement alone is not sufficient to make this determination.

TO AVOID INJURY, read, understand and follow all instructions and warnings in this manual before using your VARILITE product.

There are hazards associated with the use of wheelchairs and adaptive equipment. If you misuse your wheelchair or your back support you can be seriously injured.

#### ! WARNINGS

- Installation, adjustment and any modification of your VARILITE back support should be performed only by a qualified seating professional.
- Ensure the cushion and cover are free of foreign objects that could damage the cushion or cause increased localized pressure.
- Never use your VARILITE back support if any hardware is missing, malfunctioning, or broken. Seek qualified assistance with any repairs or adjustments.
- Never exceed the weight capacity of the wheelchair as specified by the manufacturer.
- Never exceed the weight capacity of the back support.
- Never perform weight training or other activities of daily living that cause you to exceed the weight capacity of the wheelchair.
- Never lift the wheelchair by the back support.
- Never store your back support in conditions of extreme heat or expose to direct sunlight for long periods of time. Back support surfaces may become extremely hot if left in the sunlight or in a hot car.
- Never use the back support without its cover. Using the back support without its cover increases the risk of flammability. When the cushion and cover are used together, VARILITE back supports have met the flammability requirements of ISO 7176-16 and ISO 8191-1.
- Never leave Slide Caps in the open position. Doing so may allow the Slide Brackets to disengage from the Holsters, causing the back support to fall off of the wheelchair. This can result in serious injury. *Fig. A*

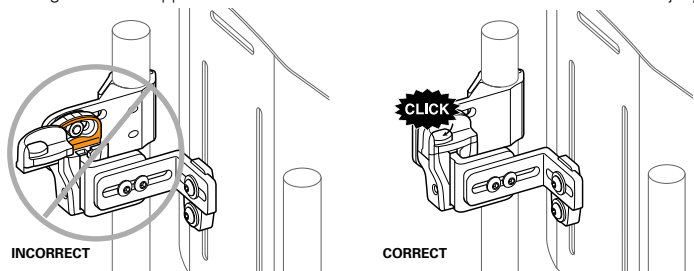
Fig. A

### Symbol Glossary

	Weight Limit
	Warning / Caution
	Authorised Representative in the European Union
	Width of product
	Depth of product
	Bottom or inferior orientation of product
	Front or anterior of product
	Back or posterior of product
	Manufacturer
	Date of Manufacture
	Consult Electronic Instructions for Use
	Solution consisting of minimum 70% Isopropyl Alcohol.

**⚠ WARNINGS**

- The Icon back system is designed to be used only with round wheelchair backrest cane diameters equal to 1 in (25mm) or less. Wheelchairs are not required to have stabilizer bars, provided the canes can maintain their position. Failure to use compatible equipment may result in damage to your back system, wheelchair, and serious injury.
- Thoroughly clean canes before installing shims. Grease, oil or other residue on the canes may result in shim slippage. Shim slippage may result in equipment breakage and serious injury.
- Failure to install and use Cane Bracket Shims as instructed can result in equipment breakage and serious injury.

**MOTOR VEHICLE TRANSPORTATION SAFETY WARNING:**

**⚠** It is safest to transfer out of your wheelchair and into a manufacturer-installed vehicle seat, and use the vehicle's crash-tested occupant restraint system. Failure to use the appropriate equipment for restraint in a vehicle can result in serious injury or death.

- The ANSI RESNA WC-20 and ISO 16840-4 standards are not considered applicable to individual seating system components. These standards address complete seating systems consisting of a seating surface and a back support used together in a specific configuration.
- An 18" wide Icon Tall back support and an 18" wide by 18" deep Evolution seating system were configured onto a surrogate wheelchair frame and successfully crash tested according to the test specified in ANSI RESNA WC-20.
- A risk assessment should be performed by competent seating professionals in the case that a provider wishes to configure VARILITE products for use during transportation in a motor vehicle.

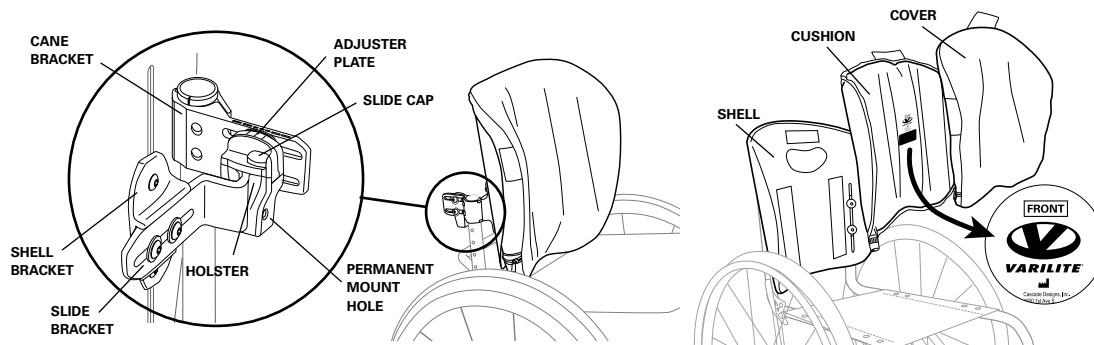
**⚠ CAUTIONS**

- The Icon Back Mid and Icon Back Tall are compatible with the VARILITE PAL and Fixed Lateral Supports. If your back support was configured with lateral thoracic supports (Laterals), see the instructions for use that came with the laterals for specific safety and operation information.
- To ensure optimal cushion performance and foam integrity, VARILITE recommends that you leave the valve open and allow your VARILITE back cushion to fully inflate overnight at least once a week. Repeat the "Cushion Setup and Adjustment" procedure when you are ready to use it again.

**Back Support Initial Configuration**

**COMPONENTS**

Please familiarize yourself with the parts of the back system before installation, adjustment or use.

**ASSEMBLY AND MOUNTING**

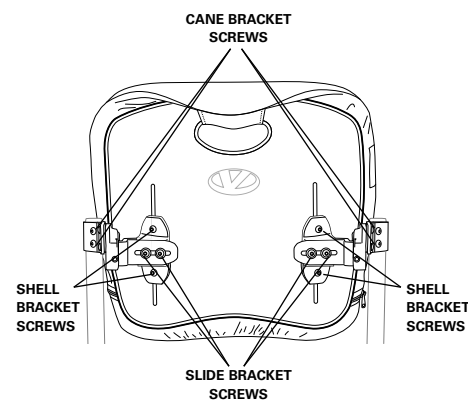
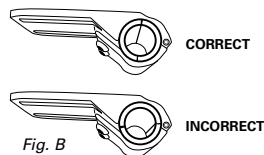
Adjustment may be necessary to complete the installation and accommodate the width of the wheelchair, or for desired back support position.

**The following steps must be done before the client is in the wheelchair:**

1. Determine the optimal height of the back of the wheelchair.
2. Attach Cane Brackets to each wheelchair cane, ensuring that the Holsters face each other and the slide caps open rearward.
3. Align the two Cane Brackets so they are parallel and level with each other, tighten Cane Bracket Screw enough to hold the Cane Brackets in place.
4. Loosen the Slide Bracket Screws.
5. Insert the back so that the Slide Brackets fit securely into the Holsters.
6. Tighten the screws on the Cane Brackets and Slide Brackets to 4.5 Newton Meters.
7. Slide left and right Slide Caps closed, toward the Shell.

If shims are required for your installation, use the recommended size shown in the chart below and ensure shims are oriented as shown in *Figure B*:

Fitting Cane Bracket Shims		
Cane Size	Shim Size	Included with Hardware?
7/8 in. (22 mm)	1/8 in. (3 mm)	Yes
3/4 in. (19 mm)	1/4 in. (6 mm)	Yes

The following steps can be done with or without someone in the wheelchair:

**Depth and Angle Adjustments:**

1. Loosen the two Cane Bracket Adjustment Screws and adjust for desired level of depth and angle. Only loosen adjustment screw 3/4 of a turn to slide. Do not disconnect from the attached unit. *Fig. C*
2. Use reference marks on the top of the Holster to make incremental adjustments.
3. Adjust the hardware on both Cane Brackets before fully tightening the adjustment screws.
4. Tighten the two Cane Bracket Adjustment Screws to maintain the adjustment.



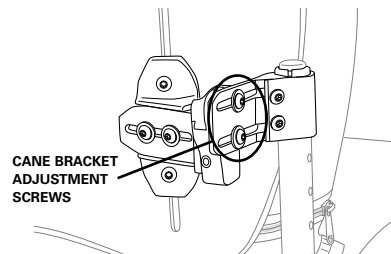
Fig. C

**Horizontal Shell Bracket Adjustment:**

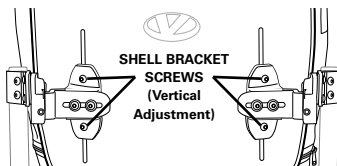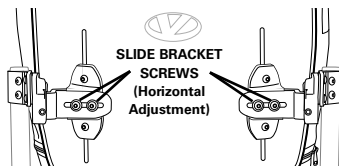1. Loosen the two Slide Bracket Screws.
2. Make lateral Shell adjustments.
3. Retighten the screws.

**Vertical Shell Bracket Adjustment:**


1. Loosen the two screws on each Shell Bracket.
2. Make vertical Shell adjustments.
3. Retighten the screws.

For additional hardware configuration options, see the Technical Guides available at [www.VARILITE.com](http://www.VARILITE.com).


**Back Support Operation**

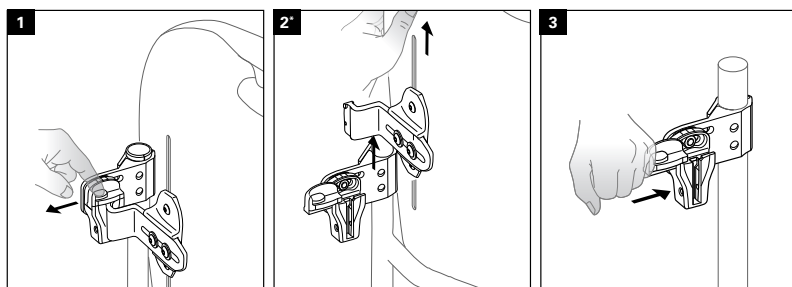**REMOVAL**

\*NOTE: When Permanent Mount Screws are engaged, Slide Brackets cannot be removed from the Holster

**INSERTION**

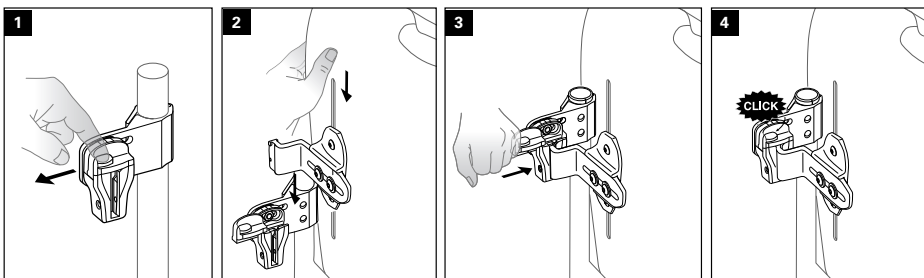**PERMANENT MOUNT INSTALLATION**

1. Loosen the left and right Slide Bracket Screws.
2. Install the Permanent Mount Screw using the 3-32 in. hex key through the Permanent Mount Hole into the Slide Bracket.
3. Tighten the left and right Slide Bracket Screws.

**CUSHION SETUP AND ADJUSTMENT:**

**Inflating Cushion:**

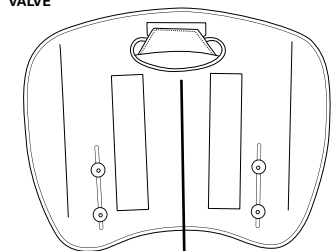
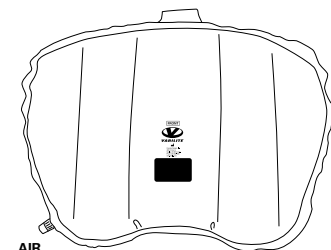
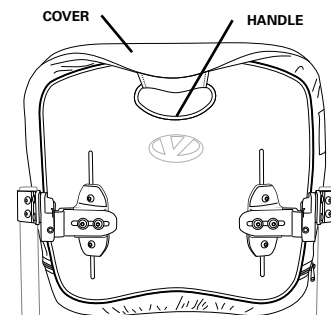
1. To inflate cushion, remove load from cushion.
2. Open air valve by turning valve counterclockwise.
3. When cushion is fully inflated, close air valve by turning valve clockwise.

**Adjusting Cushion:**

1. To adjust the air level of the VARILITE back support cushion, begin with the cushion fully inflated and the valve closed.
2. Position yourself securely in your wheelchair.
3. Turn the valve counterclockwise to open the valve and release air.
4. Close the valve when the most comfortable and supportive position is achieved.

**Removing Cushion:**

1. Pull the cushion up to detach the cover from the shell.
2. Detach the handle flap from the handle patch inside the shell and pull the flap through the slot in the shell.


3. Remove the back cushion from the attached flap on the inside of the cover.

**Note:** Installation is the reverse of removal.


### ADDITIONAL BRACKET CONFIGURATIONS

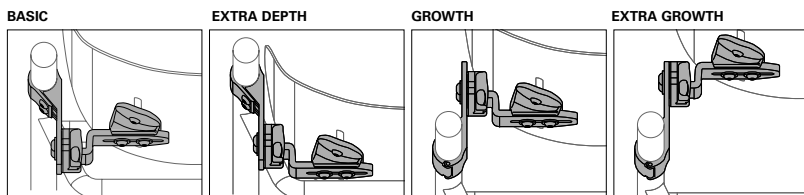The hardware is designed to support alternate configurations, shown below:

#### Icon Back

#### JUNIOR Back Support

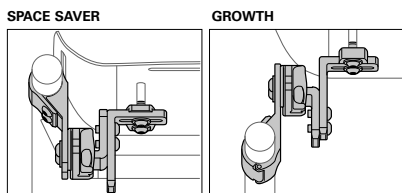
### TROUBLESHOOTING

For additional assistance, please contact VARILITE customer service.

**The VARILITE Back System should operate smoothly. The back support should remove and insert without binding. If the back support binds or seems “sticky” during removal and insertion, quick and easy adjustments will correct the situation.**

**Only loosen the Cane Bracket Screw three-quarter (3/4) turn when making depth or angle adjustments. Loosening the screw more than a three-quarter (3/4) turn will cause it to disconnect from the nut inside the Holster.**

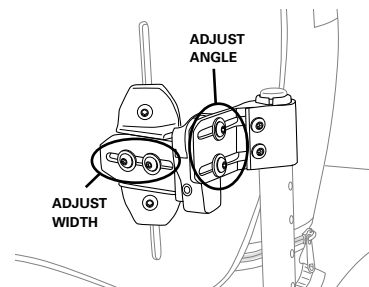1. Check the back height. Ensure that the Cane Brackets are installed at the same height and are parallel.
2. Allow the Slide Brackets to self-center. Loosen the left and right Slide Bracket screws until the Slide Brackets fit loosely and self-center in the Holsters. Tighten the Slide Bracket Screws.
3. Check the depth adjustment. Ensure that the depth adjustment is the same on the left and right Holsters. Loosen the top and bottom Cane Bracket Adjustment Screws three-quarter (3/4) turn. While holding the back system at the appropriate depth, tighten the top and bottom adjustment screws.
4. Check the angle adjustment. Ensure that the angle adjustment is the same on the left and the right Holsters. Loosen up the top Cane Bracket Adjustment Screws a three-quarter (3/4) turn. While holding the back support at the appropriate angle, tighten the top adjustment screw.

### CLEANING AND DISINFECTION

- ⚠ Warning: Do not use quaternary ammonium-, chlorine- or hydrogen peroxide-based products to disinfect your back support. These products can damage your back support.
  - ⚠ Disinfectants are NOT effective on porous surfaces such as foam or wood. If necessary, they may be cleaned by hand wiping with a damp cloth. They cannot be effectively disinfected and must NOT be used by different individuals.
  - 🧴 VARILITE recommends a minimum 70% solution of isopropyl alcohol for use as a disinfectant.
- When cleaning and disinfecting the shell, avoid getting liquid underneath the edge trim.

#### Cover Cleaning and Disinfection Procedure:

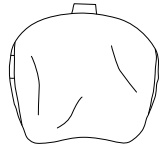
See laundering symbols in this document and on care and use tag.

#### Cushion Cleaning and Disinfection Procedure:

1. Hand wash cushion under hot tap water (40C) until all visible soil is removed from the cushion. Use mild liquid dish soap if needed. Repeat if necessary.
  2. Rinse the cushion thoroughly, so that the soap is completely removed from the cushion.
  3. Let cushion dry for at least 30 minutes.
  4. Spray entire cushion with 70% Isopropyl alcohol (IPA) ensuring that the entire pad is saturated.
  5. Let cushion dwell with 70% IPA for 10 minutes.
  6. Repeat steps four and five once more.
- Ensure cushion is fully dry before resuming normal use.


#### Shell Cleaning and Disinfection Procedure:

1. Hand wash shell with hot tap water (40C) until all visible soil is removed. Use mild liquid dish soap if needed. Repeat if necessary.
  2. To disinfect, spray with 70% Isopropyl Alcohol, ensuring all surfaces are saturated. Maintain a 10-minute wet time.
- Ensure shell is fully dry before resuming normal use.


**Cover Cleaning and Disinfecting Instructions**

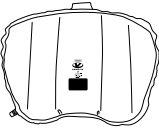**Cushion and Shell Cleaning Instructions**

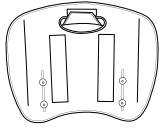**Cushion and Shell Disinfecting Instructions**

### TRANSPORTATION, DISPOSAL, AND RECYCLING

Should you need to ship your VARILITE product, place in a plastic bag to keep dirt and debris off the back support, and place into a cardboard shipping box.

At the end of its useful life, treat your VARILITE product as general waste and dispose of in accordance with all applicable local regulations. Contact your local recycling authority to determine if recycling options exist for the product.

### MANUFACTURER AND WARRANTY INFORMATION

For VARILITE product warranty information, please refer to the dealer or distributor you purchased your product from.

Products may be covered by one or more US and international patents and trademarks, including Icon™ Back System and JUNIOR Back Support products.

In the event of a serious issue or injury, please report the incident to VARILITE customer service. Additionally, if you reside in the European Union, please notify the manufacturer or distributor, and the competent authority of the Member State in which you are established.

Information printed in this manual may have changed since this document was printed. Current information is available on our website at [www.VARILITE.com](http://www.VARILITE.com).

Product Registration is available on our website at [www.VARILITE.com](http://www.VARILITE.com).

Cascade Designs, Inc.  
4000 1st Ave South  
Seattle, WA 98134 US  
Phone: (800) 827-4548  
Fax: (206) 343-5795  
[www.VARILITE.com](http://www.VARILITE.com)

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MDSS GmbH